Fit 4-All Music

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For very frequent use we recommend to choose one of our "custom made earplugs". Available in different attenuation values, filters can easily be changed by the user. Request info from your dealer or visit www.variphone.com.

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
SNR value in accordance with EN ISO 4869-2

CE EN 352-2

Sound Technology by
DYNAMIC
EAR COMPANY

Tested by: PZT GmbH | An der Junkerei 48 F
D - 26389 Wilhelmshaven | Germany | ID nr: 1974

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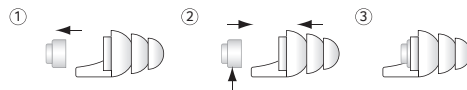
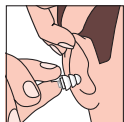
Fit 4-All Music

Universal reusable hearing protection
Linear attenuation and high sound fidelity

The Fit 4-All Music is a universal reusable earplug designed specifically for musicians and music enthusiasts. While other universal hearing protection can sound muffled and unnatural, the Music filters provide full frequency range, enjoyable audio reduced to safer listening levels. Designed primarily for music applications.

How to insert the Fit 4-All?

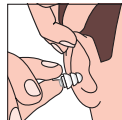Before insertion, please select the appropriate size of silicon tip (medium or mini). Both sizes are supplied with this product. The ear tips can be easily changed by pulling them off the filter nozzle and replacing them with another size.

Pull the ear a little upwards and insert the earplug in the ear. Gently make a "push and turn" move on the earplug until you get the feeling a seal of the ear canal is achieved. If no seal is achieved, please try another size ear tip.

Remove earplug by a gently "turn and pull" move to gradually break the seal with the ear canal. Please perform this gently to prevent creating a vacuum.

Cleaning & durability

The Fit 4-All are reusable earplugs. Please keep them safely stored in the pouch provided with the plugs. Pay particular attention to maintenance so that optimal performance and durability can be guaranteed. Clean them regularly with hand warm water and a bit of soap. Let dry afterwards. Do not use solvents on this product with exception of alcohol wipes. When not properly used, the attenuation values can deviate from the stated values.

In case of any uncertainty please contact Variphone.

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